




Instructions for **MINI MOUTH®** Finger Puppet

1. Place ring and middle finger in top pocket of mouth.
2. Place index finger in tongue.
3. Place thumb in bottom pocket of mouth.
4. Use knuckle for k/g.
5. Slide tongue out for th.
6. Place tongue behind teeth and close mouth for s.



For the r, l, t, and d sounds:

1. Turn *Mini Mouth®* upside down.
2. Place thumb in bottom pocket of mouth.
3. Place index finger in tongue.
4. Place ring and middle finger in top pocket of mouth.
5. Turn mouth right side up.
6. Lift tongue up for l, t, and d.
7. Lift tongue up and pull back for r.

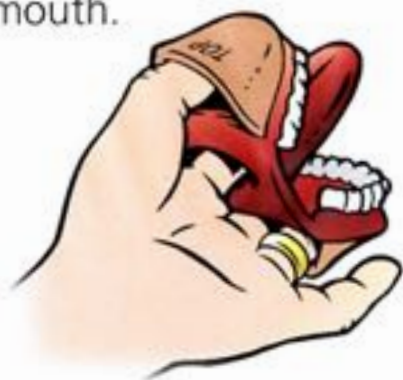

Use the *Mini Mouth®* as a model to demonstrate:

1. Tongue placement for swallowing.
2. Oral motor exercises.

Cleaning and Safety Information:

Clean the *Mini Mouth®* using antibacterial soap and water.

WARNING! This product is not recommended for those with latex allergies.